

BLEEP DREAMWAY VENT DIFFUSER APPLICATION INSTRUCTIONS



Step 1
Parts needed: DreamWay
interface, black hook & loop
filter strap, 2 white filters.




Step 3B

During the application of the completed vent diffuser strap, the soft side of the hook and loop strap needs to be on the outside of the DreamWay.



Step 2
Remove DreamWay vent swivel on end of tube from the bottom of the interface.

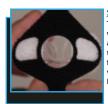

Step 3C
This is to make sure the soft side is what comes in contact with the face, as shown in the image to the left, trapping the hook and loop side with the white filter material against the actual plastic vent openings on the bottom of the DreamWay.



Step 3A

Apply the 2 pieces of white filter material over the 2 cutout side openings of the black hook and loop filter strap. Be sure that the "scratchy side" is facing up and on the side you are applying the white filter material on



Step4A
Apply the hook and loop filter strap, with both white diffuser filters in place, back onto the bottom of the vent openings, where the tube was formerly attached.





Step 4B
Make sure the white filters are in line with the vents on each side of the bottom of the DreamWay.



Step 5A
Snap the tube swivel/hose onto the DreamWay with the vent diffuser strap and filters in place.



Step 4C
Bring the ends of the black straps to the top middle of the silicone.



<u>Step 5B</u> The DreamWay is now ready for use with your PAP device.



Step 4D
Now fasten together with the hook and loop material to hold it in place through the center.



Step 6

The vent diffuser strap does not need to be removed to clean the DreamWay.
Recommended cleaning is to leave it in place on the DreamWay and wash in warm soapy water. Rinse and let air dry. White filter inserts are available for replacement each month as a separate purchase, along with the black hook and loop strap in both quarterly and annual bundles.